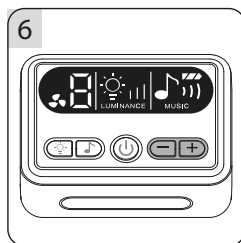
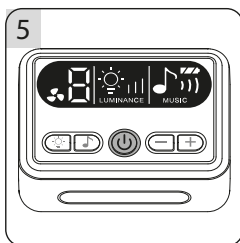
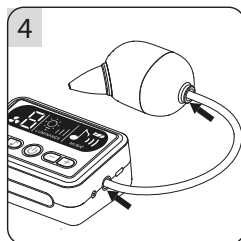
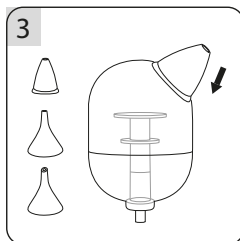
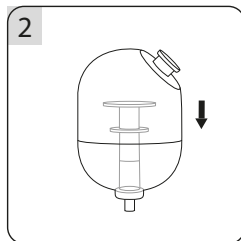
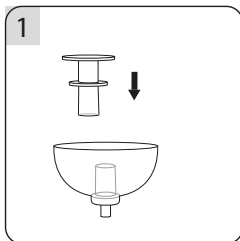
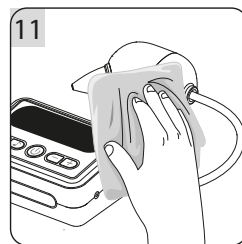
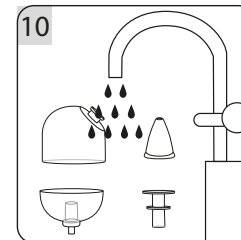
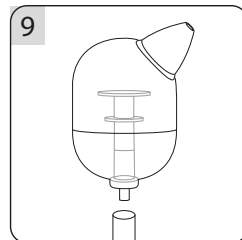
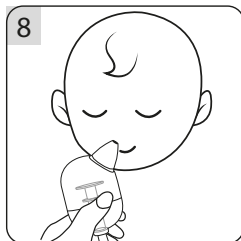
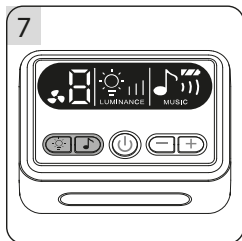
2

2

2

Baby Nasal Aspirator

Please read the instruction manual carefully before the first use. Retain for future reference.

IMPORTANT SAFETY INFORMATION

- When any abnormality occurs, turn off the device.
- Use only the tips provided by the manufacturer. Using other tips or accessories may result in improper operation or damage to the device.
- Do not turn on the nasal aspirator while charging.
- This device is intended for adult operation only.
- This product is not a toy – Keep out of the reach of children.
- Do not expose the device to high temperatures, high humidity, or direct sunlight.
- The device is designed specifically for removing nasal mucus from a baby's nose for non commercial home use only. It must not be used to clean other parts of the body.
- Keep the parts and body clean and dry after each use.
- Check the device and all its accessories regularly.
- Charge the battery to 100% every 2 months if not used for a long time.
- Do not use chargers that do not meet the required parameters to charge the battery.
- Do not use a damaged USB cable.
- During use, the baby/child should not hold their breath, close their mouth, or swallow. Excessive vacuum pressure may cause an imbalance in ear pressure.
- Do not exert excessive force when placing the suction tip into the nasal cavity.
- If the baby/child experiences any discomfort while using the device, discontinue use immediately and consult a doctor.
- When playing a melody, do not place the device close to baby's head.
- It is recommended to set the sound level of the played melodies to the lowest possible level.
- Placing the loudspeaker of the nasal aspirator close to baby's ears may be harmful to the child due to the extensive noise.
- When transporting by plane, take the device as hand luggage due to the built-in battery.

Product Features

- Efficiently cleans nasal passages with comfort and safety. Nine suction levels for different needs. If baby has a big and wet snot, may be Lv1 can aspirated it easily. If baby has a small and dry snot, then may be need to use Lv9. It needs to be adjusted depends on the real conditions.
- Built-in calming melodies and gentle lights.

Parts of the device fig.1.1

- | | |
|---|--|
| 1. Main body, | 11. Suction control buttons “+” & “-”, |
| 2. Silicone nozzle, | 12. Melody and volume control button, |
| 3. Light/brightness indicator, | 13. Power on/off button, |
| 4. Suction level indicator, | 14. Air intake port, |
| 5. Light and brightness control button, | 15. Air outlet port, |
| 6. Battery power indicator, | 16. Collection cup. |
| 7. Melodie/volume indicator, | |
| 8. USB charging port, | |
| 9. Silicone drain tube, | |
| 10. Night light, | |

3

Accessories fig.1.2

1. Collection cup lid,
2. Collection cup insert,
3. Collection cup bottom cover with gasket,
4. Air tube,
5. USB charging cable,
6. Tweezers,
7. Conical nozzle,
8. Flat nozzle,
9. Angled nozzle,
10. Cleaning brush.

Note: Please note that before using a nasal aspirator, make sure to understand the characteristics of each nozzle (see below) and choose the appropriate one based on your child's age to ensure a comfortable and effective user experience.

Nozzle Type	Depth	Age Range
Conical Nozzle 7	Shallow	0–2 years old
Flat Nozzle 8	Deep	0 years and above
Angled Nozzle 9	Deepest	0 years and above

Before First Use

- It is recommended to disinfect the silicone nozzle before use by any method.
- Collection cup pour with boiling water. Do not sterilize.
- Do not use high-temperature devices such as microwaves for disinfection.
- Before using, please check the product for proper sealing and ensure all accessories are complete and properly sealed.
- When using this device, please ensure that the child maintains a seated position with the nostrils facing downward. Avoid tilting the head back.
- It is important to keep the baby's mouth open during use to ensure proper airflow in the nasal cavity and prevent any discomfort caused by negative pressure.
- Avoid obstructing the baby's other nostril while operating the nasal aspirator to prevent any discomfort resulting from negative pressure in the head.
- When the collection cup is halfway full, it is recommended to clean it promptly before continuing to use, as this prevents nasal mucus from being drawn into the pump and potentially damaging the product.
- Please refrain from immersing or rinsing the entire product in water during cleaning, as it is not waterproof.

Assembly instructions fig.2.

1. Install the collection cup insert to collection cup bottom cover fig. 2.1.
2. Attach the collection cup onto the bottom cover fig 2.2.
3. Choose the nozzle (conical, flat, angled) and install it onto the collection cup fig. 2.3.
4. Connect the collection cup to the main body's air intake port (suction) using the tubing fig. 2.4. Ensure that the tube is fully inserted.
5. Long press the power button to turn the device on fig. 2.5.
6. Use '+' or '-' to adjust suction strength to adjust the suction level fig. 2.6.
7. Use music/light functions if needed fig. 2.7.

8. Gently insert the nozzle into the baby's nose fig. 2.8.
9. After cleaning your baby's nasal mucus, remove the collection cup fig. 2.9.
10. Rinse components promptly and dry them. NOTE!!! Do not wash main body fig. 2.10.
11. After cleaning, use a soft cloth to dry the surface of the device and store it properly in a cool dry place, avoiding direct sunlight fig. 2.11.

NOTE: For better results, if the baby's nasal mucus is dry, it is recommended to use saline nasal spray to moisten it before using the electric nasal aspirator. If you see that access to the nose is difficult due to dried mucus, you can gently use tweezers to unblock it.

How to use

- Power Switch  Long press to turn the device on/off. Short press to pause working.
- Melody and volume control button  Short press to adjust the volume to levels 1 to 3. The button sequence is: 1 --> 2 --> 3 --> Off --> On. Pressing again will switch to another melody.
- Night Light Button  Long press to on/off during working on not working; single press to adjust brightness levels 1 to 3.
- Suction Power Buttons: Press '+' or '-' to select suction levels from 1 to 9. You can use them depending on needed suction levels.
- The device has a memory function. It starts with level 3. You can choose different when needed.
- The device features an automatic shut-off function. After switching on the aspirator (by pressing and holding the button), then pressing it briefly again and leaving it in this state, it will automatically turn off after 8 minutes — the display will turn off as well.

Cleaning and Storage

- Always turn off the power after use.
- Remove the suction tip and collection cup and clean them with mild detergent, water and rinse thoroughly after each use. Dry. You can sterilize the tips using any method. Do not sterilize the collection cup. You can pour with boiling water and then dry it thoroughly.
- Keep the parts and body clean and dry after each use.
- The main body of the product is not waterproof, so do not rinse it with water or clean it at high temperatures. Please use a soft, damp cloth to wipe the main body for cleaning.
- Use added cleaning brush for better effects, to clean hard to reach areas.
- Store in a dry place, away from heat sources, direct sunlight and humid environment.

Product Specification

Model: 20/121
 Charging Voltage: DC 5V/1A
 Maximum charging current: 615 mA
 Charger power: 10W
 Built-in battery: 3.7V/2600 mAh
 Charging Time: Approx. 3 hours
 Suction Strength: 65Kpa ±3Kpa
 Intensity Levels: 9
 Maximum intensity of the melody being played (distance of 1 meter): 65 dB(A)

Intensity Levels of sound: 3
Net Weight: 276g
Product Size: 105mm x 75mm x 51mm

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. This product and its information meet the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures. The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class III appliance

Troubleshooting

To ensure the optimal performance and longevity of your baby's nasal aspirator, it's essential to address common concerns and know how to properly maintain the device. Here are some frequent issues and solutions:

- Device not turning on or turning off after a few flashes: Charge fully before use.
- Low suction or ineffective operation: Check if parts are installed properly and avoid sealing the nozzle completely against the skin.
- Hard or dried mucus difficult to remove: Use saline solution to soften before use.
- Mucus spraying from air outlet: Turn off the device, clean the collection cup, and avoid overfilling.

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Manufacturer: **Canpol Sp. z o.o.**

Puławska Street 430, 02-884 Warsaw, Poland

Product name: **Electric Nasal Aspirator for babies and Children**

Brand: **Canpol babies**

Model: **20/121**

In relation to the described product, the manufacturer declare under the sole responsibility that the product to which this declaration refers is in line with the following European Union directives:

2014/30/EU Electromagnetic compatibility (EMC)

2011/65/EU Restrictions of the use of certain hazardous substances in electrical and electronic equipment (RoHS)

2023/1542/EU concerning batteries and waste batterie

Battery type: portable battery, built into the device,

not removable by the user

Battery model: Li-Ion Battery Hao Xin 18650; 26000mAh

Nominal Voltage: 3.7V

Rated Capacity: 26000mAh

Rated Energy: 9.62Wh

Batch code: YYYY.MM

For which laboratory tests were performed in accordance with the standards:

IEC62321-3-1:2013, IEC62321-5:2013, IEC62321-4:2013+A1:2017, IEC62321-7-

1:2015, IEC62321-7-2:2017, IEC62321-6:2015, IEC62321-8:2017

Test report no. NCT240014025XR1-1 dated 09.04.2024 (NCT Technology Co. Ltd.)

The product is in line with harmonized standards and other technical standards:

- EN 55014-1:2017 + A11:2020 (EN 55014-1:2021 – not harmonized)
- EN 55014-2:1997 + A1:2001 + A2:2008 + A3:1997 (EN 55014-2:2021 – not harmonized)
- EN IEC 63000-2018
- EN 60335-1-1:2012+A11:2014+A13:2017+A1:2019+A2:2019+A14:2019+A15:2021+A16:2023
- EN IEC 61000-3-2:2019/A1:2021
- EN 61000-3-3:2013/A2:2021

The last two digits of the year in which the marking was applied

: 25

Warsaw, 19.08.2025

(place and date of issuing the declaration)

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(name, surname, position and signature of the authorized person)

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