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CHARACTERISTICS:

Innovative, patented system of pressure relief enables proper pressure distribution which prevents skin abrasions and bedsores.

Breathable, easy to clean cushion cover provides optimal comfort of use.

Anti-slip pad maintains the cushion on place.

Pressure regulation allows to change cushion height and softness.

Cushion available in three sizes:

- 38 cm x 40 cm x 8 cm
- 40 cm x 40 cm x 8 cm
- 45 cm x 40 cm x 8 cm

INDICATIONS:

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.

CONTRAINDICATIONS:

Bedsore V stage.

DIRECTIONS FOR USE:

1. Take the cushion out of the box and make sure that the valve is in open position, wait 30 seconds and the cushion will inflate automatically, without any additional devices.
2. When fully inflated, the cushion can support max 180 kg.
3. Close the valve by turning it counter-clockwise. The white line should reach the white dot.
4. In order to regulate softness and height of the cushion, sit on the cushion and turn the valve clockwise.
Adjust the cushion softness to individual needs to obtain maximum comfort.

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVICE

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ATTENTION

Never deflate completely the cushion while sitting on it. After choosing the proper softness of the cushion, close the valve (turn it counter-clockwise) so that the white line would reach the white dot- it will prevent air from escaping.

ATTENTION:

1. Avoid using the cushion in extremely high or low temperature, do not expose it to UV rays.
2. Keep away from fire.
3. Keep away from sharp objects.
4. The cushion can be used only with original cushion cover.

MAINTENANCE:

The cushion cover should be washed up by hand in warm water using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron.

Close the valve before washing the cushion. The cushion should be washed up by hand in warm water using mild detergents. Dry in a distance of artificial sources of heat. Do not iron. Always put on the cover when the cover and cushion are completely dry.

COMPOSITION:

Cushion: visco-elastic polyurethane foam, 75 D polyester, waterproof
Cushion cover: 100% polyester
Max. user weight: 180kg