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CHARACTERISTICS:

Anti-bedsore cushion consist of numerous, elastic, air interconnected cells provide proper pressure distribution which prevents skin abrasions and bedsores. Inflating and pressure control is possible due to hand pump and air valve.

Cover made of breathable, perforated fabric and empty spaces between the cells provide optimal comfort of use and air flow - warmth is taken away from the body which prevents excessive perspiration. The cushion is easy to maintain, highly hygienic.

Additionally, the cells in two-zones version are divided into two zones. Each of the zone has separated valve. Thanks to such construction it is possible (depending on cushion position- transverse or longitudinal) to correct patients position or adjust tilt angle.

Anti-slip pad maintains the cushion on place.

Cushion available in the following sizes:

- 36 cm x 40 cm x 6 cm
- 40 cm x 40 cm x 6 cm one and two-zones version
- 46 cm x 40 cm x 6 cm
- 40 cm x 40 cm x 8 cm one and two-zones version
- 40 cm x 40 cm x 10 cm one and two-zones version
- 46 cm x 40 cm x 10 cm

INDICATIONS:

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.

CONTRAINICATIONS:

Bedsores V stage.

DIRECTIONS FOR USE:

1. Take the cushion out of the box and twist the valve counter-clock wise to open.
2. Attach silicon hose of pump to valve. Inflate the cushion with pump.
3. Close the valve by turning it clockwise.

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVICE

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4. In order to regulate softness and height of the cushion, sit on the cushion and turn the valve clockwise.
Adjust the cushion softness to individual needs to obtain maximum comfort.
5. Readjusting the cushion pressure is recommended every 10 days.
6. In case of two-zones cushions, repeat actions 1-4 for the second valve.

ATTENTION:

1. Avoid using the cushion in extremely high or low temperature, do not expose it to UV rays.
2. Keep away from fire.
3. Keep away from sharp objects.
4. The cushion can be used only with original cushion cover.

MAINTENANCE:

For hygiene reason anti-bedsore cushion should be used only by one patient.

The cushion cover should be washed up by hand in warm water using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron.

Close the valve before cleaning the cushion. The cushion should be washed up with the damp cloth using mild detergents. Dry in a distance of artificial sources of heat. Do not iron.

Always put on the cover when the cover and cushion are completely dry.

COMPOSITION:

Cushion: PCV
Cushion cover: 100% polyester
Max. user weight: 130kg